

NOXYPure INSTRUCTIONS FOR USE

1

Prepare treatments by setting aside a disposable bootie, pouch of gel and disposable towel, for each foot that will be treated.
Two feet: two of each item, one foot: one of each.



2

Empty the entire contents of one pouch into each disposable bootie.



3

Insert foot into one disposable bootie, then the other if doing both.



4

Gently massage the gel onto each foot to ensure full immersion in gel.



5

Treatment application time is 20 to 30 minutes.



6

Remove feet from booties.



7

Wipe off remaining gel with the disposable towels and carefully dry feet.



8

Dispose of gel and disposable booties in your waste basket (do not pour in sink).





Therapeutic Gel with NanoBubbles of CO₂ and O₂

Intended Use:

This product is intended to relieve some of the symptoms of peripheral nerve pathology. Several treatments are usually necessary to observe significant and lasting relief. At the beginning of the treatment, 1 treatment every other day is recommended for 6 to 10 treatments. Once symptoms are stabilized, treatments can be repeated periodically as required to maintain symptoms relief.

Instructions for Use:

1. Prepare treatment by setting aside two disposable booties, 2 pouches of gel and 2 dry disposable towels.
2. Empty the entire container contents into the disposable bootie.
3. Insert foot into the disposable bootie.
4. Gently massage the foot to ensure full immersion in gel.
5. Treatment application time is 20 to 30 minutes.
6. Remove foot from booties.
7. Wipe off remaining of gel and dry foot carefully with a towel.
8. Dispose of gel and disposable booties in your waste basket.

Precautions:

1. Do not use if there is any open wound or skin breach.
2. Do not microwave
3. For external use only. Do not ingest.
4. If you observe redness or irritation, stop the treatment and contact your physician.
5. Do not walk during treatment with booties on.
6. Feet will be slippery after treatment. Make sure you dry them properly before walking.